


# NuMask<sup>®</sup>


*It's a matter of life and breath.*

## **IOM<sup>®</sup>/OPA** Lateral Grip


*Performed with provider at the side of the patient.*

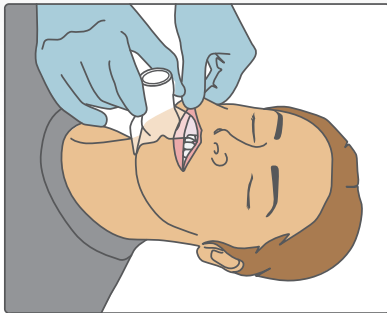


**STEP 1:** Elevate lateral edge of the lips and slide in a flap of the IOM.

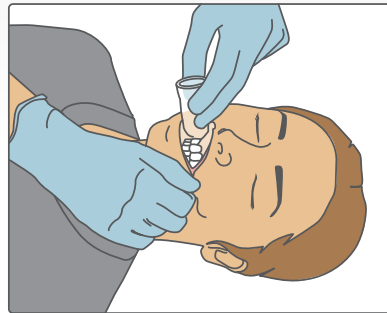


**STEP 2:** Grasp and elevate the other side of the lips to insert remainder of IOM.

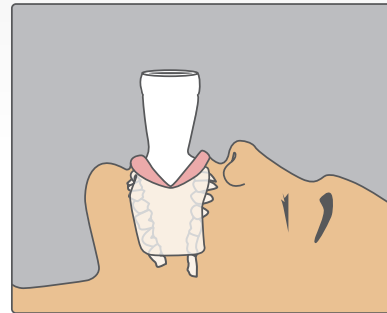


**STEP 3:** IOM sits in front of teeth & behind lips.

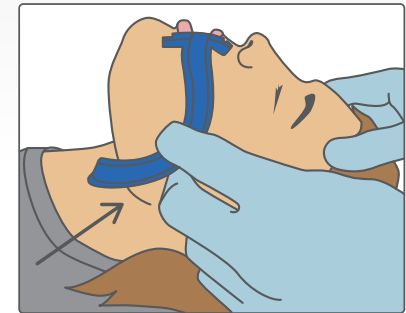


**STEP 4:** A NuMask OPA may be used if necessary to maintain the airway. First measure the OPA from the front teeth to angle of mandible.

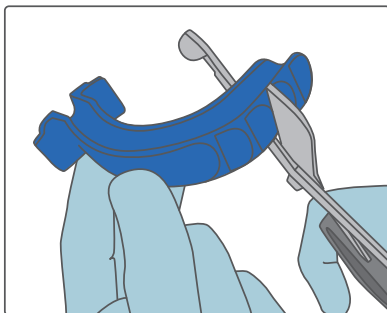


**STEP 5:** If necessary, trim the OPA to size.

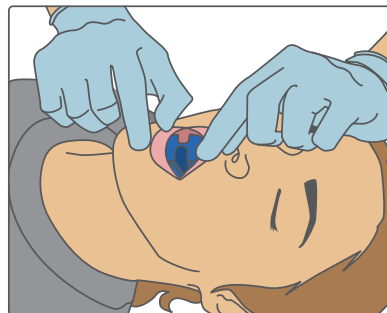

**STEP 6:** Insert and invert the OPA so that the tab sits in front of teeth and behind lips.

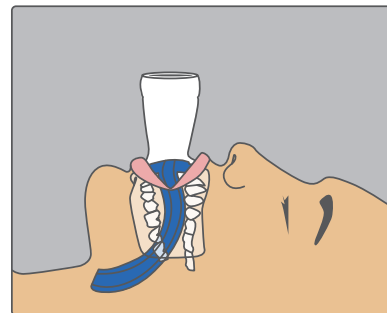

**STEP 7:** IOM is then inserted over the OPA.

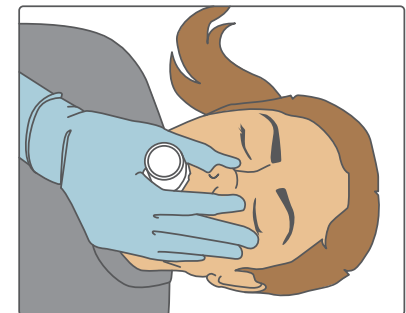

**STEP 8:** Fingers and hands surround and close the lips around the stem of the mask. Pinch the nose closed.